



JJASPP ENGINEERING SERVICES, LLC

511 Harmon Terrace, Arlington, TX 76010



817-465-4495 (office)

www.jjaspp.com

JJASPP

ENGINEERING SERVICES

AUTOCAT2/3 SERIES Balloon Pump Restraint Systems

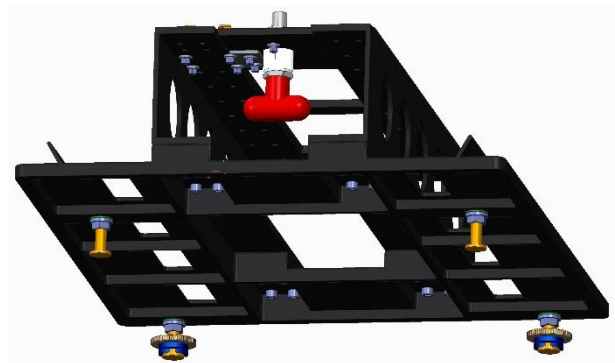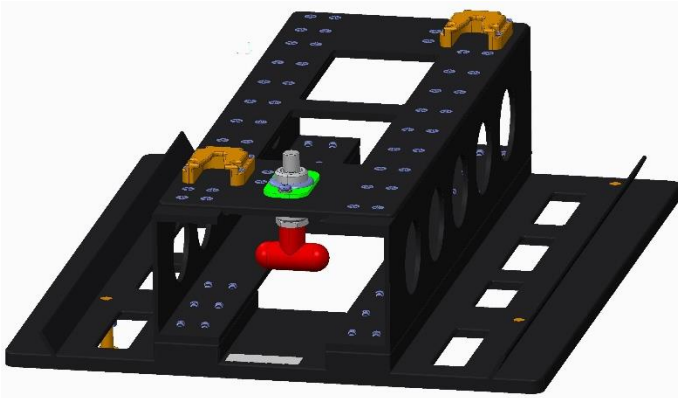

Specifications:

Weight – 12lbs.

Dimensions – W: 17", L: 20 ½", H: 7 ¾"

Certification:

The JJASPP Air Medical BPRS is certified in the following aircraft:

- | | |
|--------------------------|-------------------------|
| 1) Bell 407 series | 6) Pilatus PC-12 series |
| 2) Bell 206 series | 7) EC130 series |
| 3) Airbus EC 145 / BK117 | 8) Bell 429 |
| 4) Airbus EC135 series | 9) King Air B200 series |
| 5) Airbus AS350 series | 10) Airbus E155 series |

This mounting system has also been qualified to DOT ground ambulatory conditions pursuant to Title 49 Part 571.208 & Part 571.214 in coordination with Arrow International Balloon Pump manufacturer.





JJASPP ENGINEERING SERVICES, LLC

511 Harmon Terrace, Arlington, TX 76010

817-465-4495(office)

www.jjaspp.com

JJASPP AIR MEDICAL AUTOCAT3 SERIES BPRS

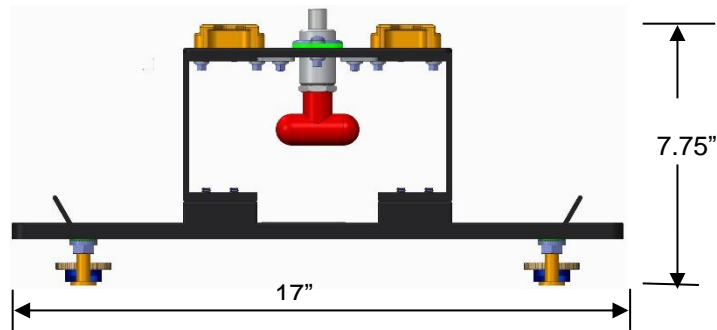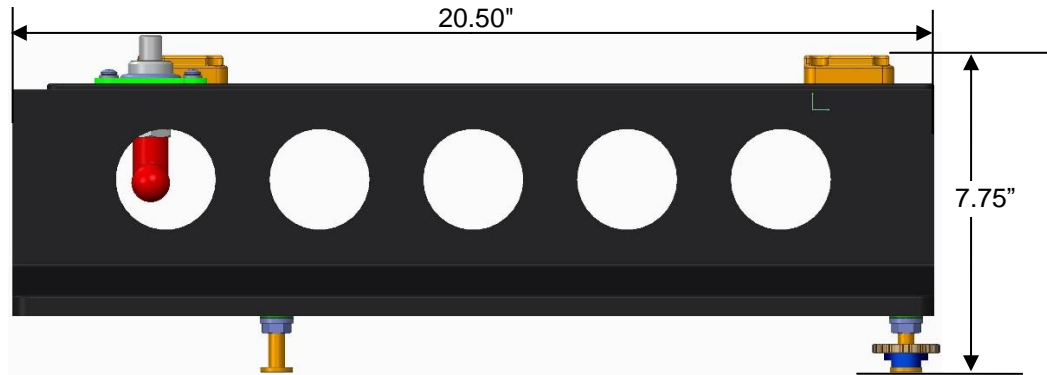

Specialized machined Restraint Fittings to avoid electrical component interference.






JJASPP ENGINEERING SERVICES, LLC

511 Harmon Terrace, Arlington, TX 76010

817-465-4495(office)

www.jjaspp.com

JJASPP AIR MEDICAL AUTOCAT3 SERIES BPRS INSTALLATION

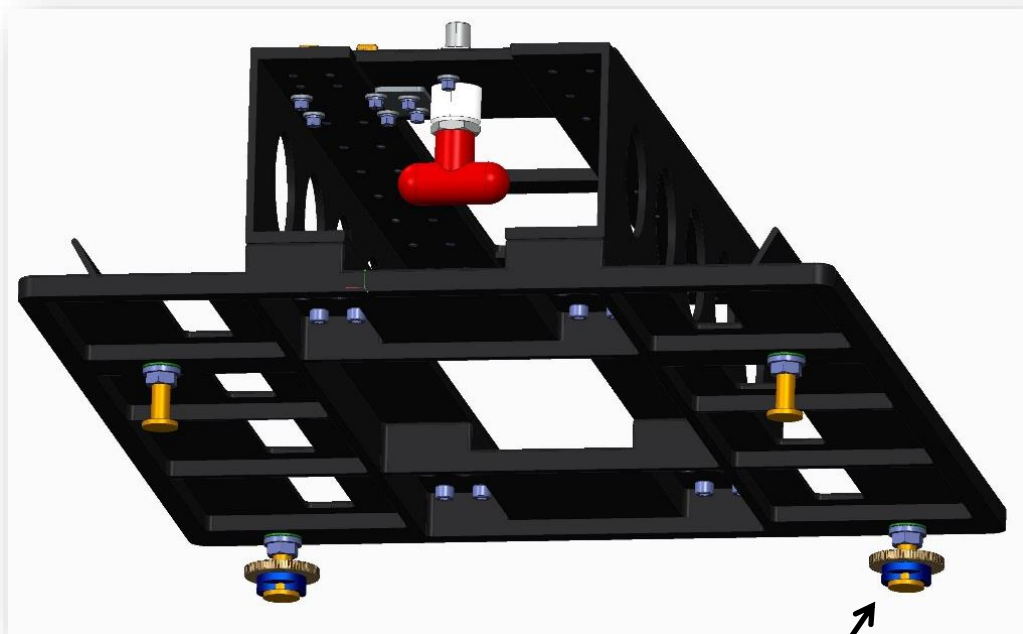

Knoll Nut
Retainer
Seat Track Cleat

1. Line-up cleats on pump mount with slots on seat tracks.
2. Bottom out seat track cleats in seat track.
3. Slide mount slowly forward until retainer fits down onto seat track between slots.
4. Tighten knoll nuts to secure retainers.




JJASPP ENGINEERING SERVICES, LLC

511 Harmon Terrace, Arlington, TX 76010

817-465-4495(office)

www.jjaspp.com


JJASPP AIR MEDICAL AUTOCAT3 SERIES BPRS

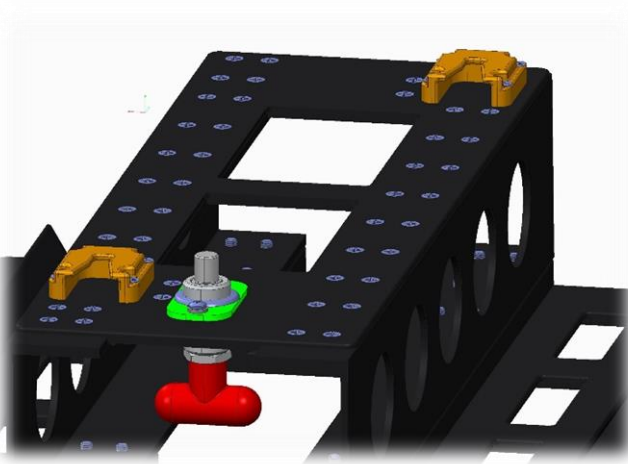

Side View of Mount



Bottom View of Balloon Pump

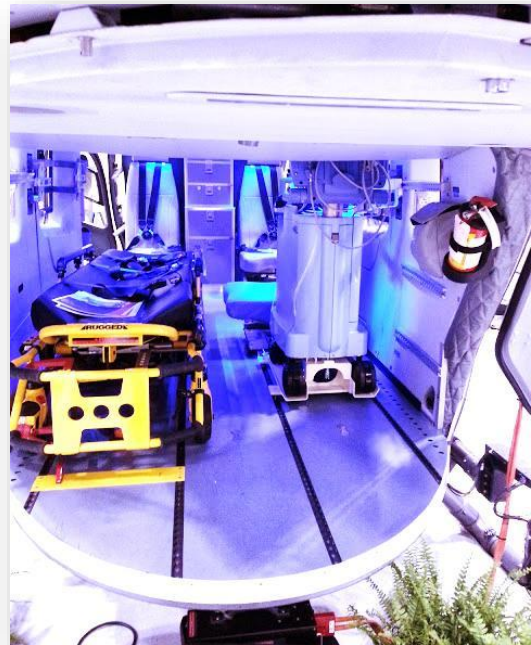

EC145 Install View