




# JJASPP ENGINEERING SERVICES, LLC

## JJASPP-MQ-CHRS Data Sheet

511 Harmon Terrace Arlington TX

817-465-4495

## CARDIOHELP Restraint System (CHRS)

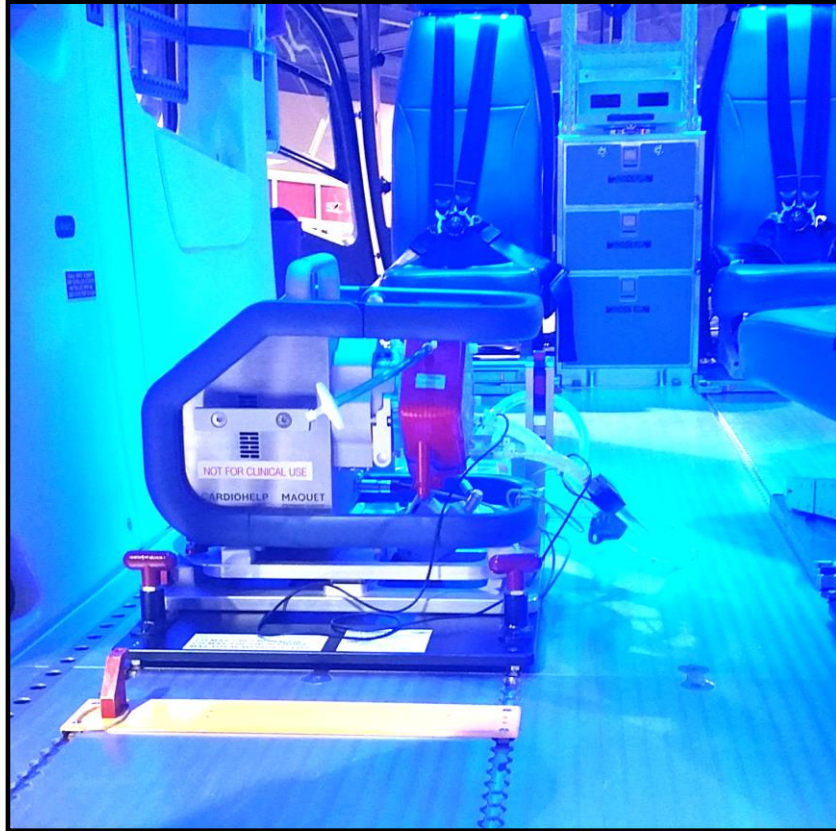

### Specifications:

Weight – 5.8 lbs.

Dimension – W: 16 3/4", L: 17 3/4", H: 1 1/4"

### FAA/EASA Certification:

The JJASPP Air Medical CHRS is certified in the following aircraft:

- 1) Airbus BK-117 – MQ40      Model A
- 2) Airbus EC135 – MQ40      Model A
- 3) Airbus EC130 – MQ40      Model B
- 4) Airbus AS350 – MQ40      Model A
- 5) Agusta A119 – MQ40      Model C
- 6) Agusta AW119 MKII – MQ40 Model C
- 7) Pilatus PC12 series
- 8) Beechcraft King B200 series
- 9) Bell 206, 407, & 429

## INSTALLATION

### Cardiohelp Mount Installation Instructions

1. Line-up cleats on mobile holder with slots on CHRS track.
2. Bottom out cleats with base on track.
3. Slide mobile holder slowly forward
4. Unlock Quick Release Plunger so it can engage down into slot on track, which secures mount from sliding forward or backward.
5. Finally, at the same time plunger is engaged into slot, the cleats should be in the locked position under the flange and in between slots, thus securing the mount from coming upward.

