



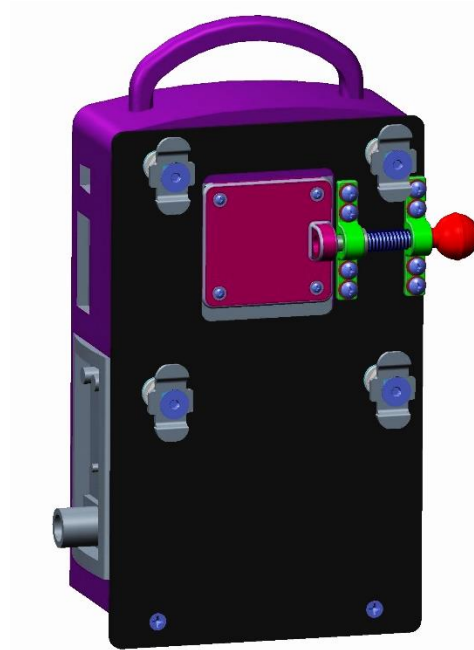

JJASPP-VRS10 Data Sheet

Revel Ventilator Restraint System (VRS)

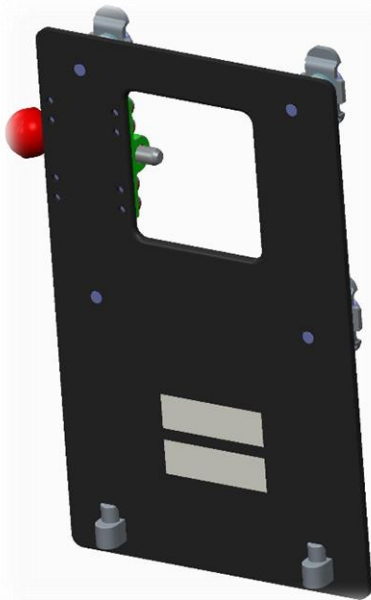

Specifications:

Weight – 9 lbs.

Dimension – W: 16 3/4", L: 17 3/4", H: 3 1/4"

FAA/EASA STC Certification:


The JJASPP Air Medical BPRS is certified in the following aircraft:

- 1) Airbus EC145 / BK-117 – MQ10-3
- 2) Airbus EC135 – MQ10-3
- 3) Airbus EC130 – MQ10-2
- 4) Airbus AS350 – MQ10-3
- 5) Agusta A119 – MQ10-4
- 6) Agusta AW119 MKII – MQ10-4
- 7) Bell 407 – MQ10-5
- 8) Bell 429 – MQ10-3
- 9) Pilatus PC-12 – MQ10-3
- 10) Pending Certification: Beechcraft King Air





JJASPP MDRS VRS10

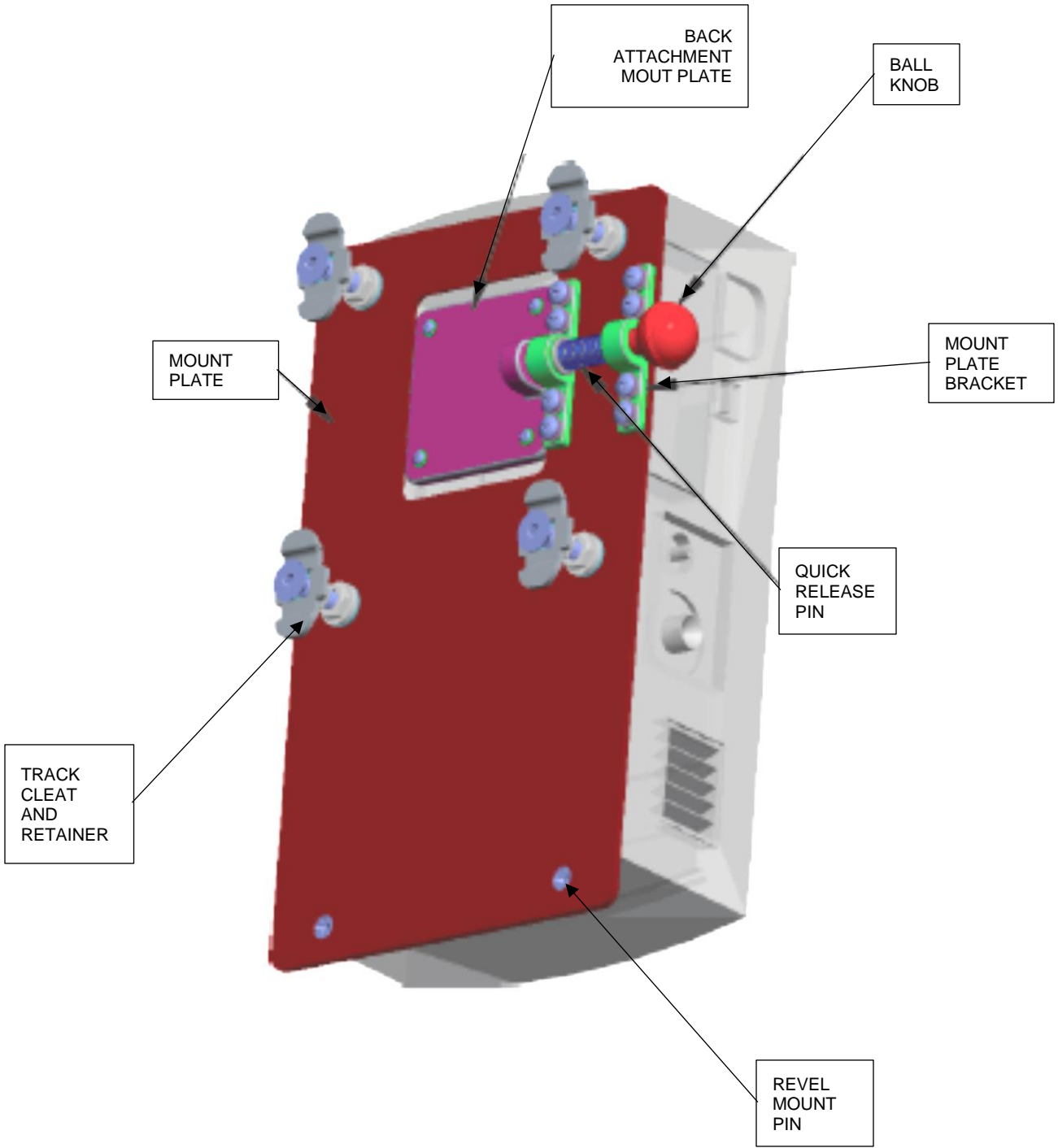



INSTALLATION

VRS Installation Instructions


1. Line-Up the Cleats on the VRS with the Slots in the Seat Tracks.

2. Bottom Out Cleats with Base on Track.


3. Slide Slowly Sideward, Retainer will Drop Down onto Track and Engage in between Slots.

4. Finally, Tighten Nut to Retainer IAW AC43-13 Section 7 Chapter 4.0 to Secure Retainers in Place

5. The VRS Should Now be Fully Engaged and Locked in the Aircraft Seat Tracks.

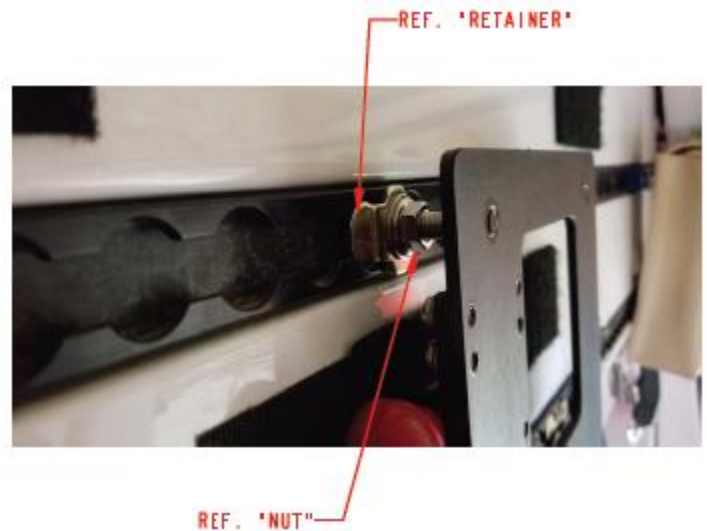

Cleat in locked position
beneath track flange and
between track slots.

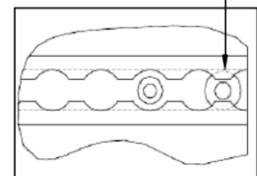



REVEL VENTILATOR MOUNTED IN VRS

VRS Installation Instructions (cont'd)

6. Align Holes on the Bottom of the Revel Ventilator with the Mount Pins
7. Slide the Ventilator Downward Until Bottom Slots on the Revel Fully Engage with the Mount Pins.
8. Pull the Ball Knob Outward, then Lean the Ventilator Back Flush Against the Mount Plate.
9. Release the Ball Knob Engaging the Quick Release Pin to Secure the Ventilator to the Mount Assembly.
10. Ensure that the Ball Knob is Full Seated Against the Mount Plate Bracket.

