


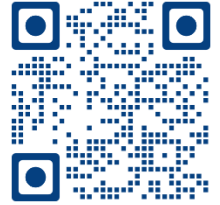

JJASPP ENGINEERING SERVICES, LLC  
JJASPP Medical Device Restraint System  
JJASPP-IP10-5/8 Data Sheet



**Infusion Pump Restraint System  
(IPRS)  
For Qcore Sapphire Triple Infusion Pump Mount**



**Specifications:**

Weight – 1.5 lbs.

Dimension – W:12.25", H: 3.81", D: 4.42"

**Certification:**

The JJASPP MDRS is either approved or pending imminent certification in the following aircraft:


- 1) Airbus BK-117
- 2) Airbus EC155
- 3) Airbus EC135
- 4) Airbus EC130
- 5) Airbus AS350
- 6) Pending Certification: Agusta A119, A109
- 7) Agusta AW119 MKII
- 8) Pending Certification: Bell 206, 407, & 429
- 9) Pilatus PC-12
- 10) Beechcraft B200




JJASPP ENGINEERING SERVICES, LLC  
JJASPP Medical Device Restraint System  
JJASPP-IP10-5/8 Data Sheet



IPRS installed on FAA Approved EMS Medical Wall

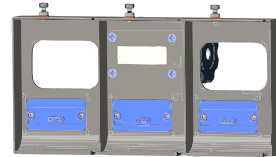

IPRS without CME Infusion Pump Installed

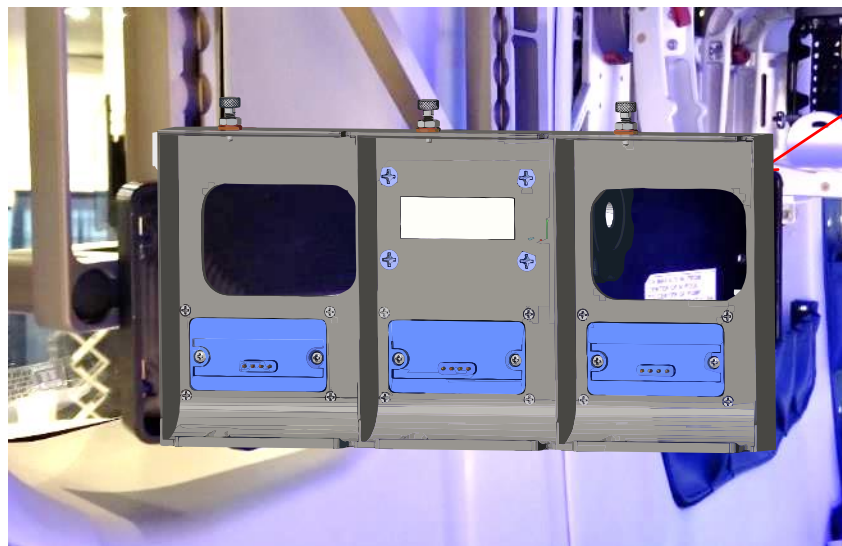

Main Housing




JJASPP ENGINEERING SERVICES, LLC  
JJASPP Medical Device Restraint System  
JJASPP-IP10-5/8 Data Sheet




IPRS Main Housing Connection

IPRS has a 30° Vertical Range rotation capability, a 45° Horizontal Range rotation capability and a 180° Vertical tilting capability.

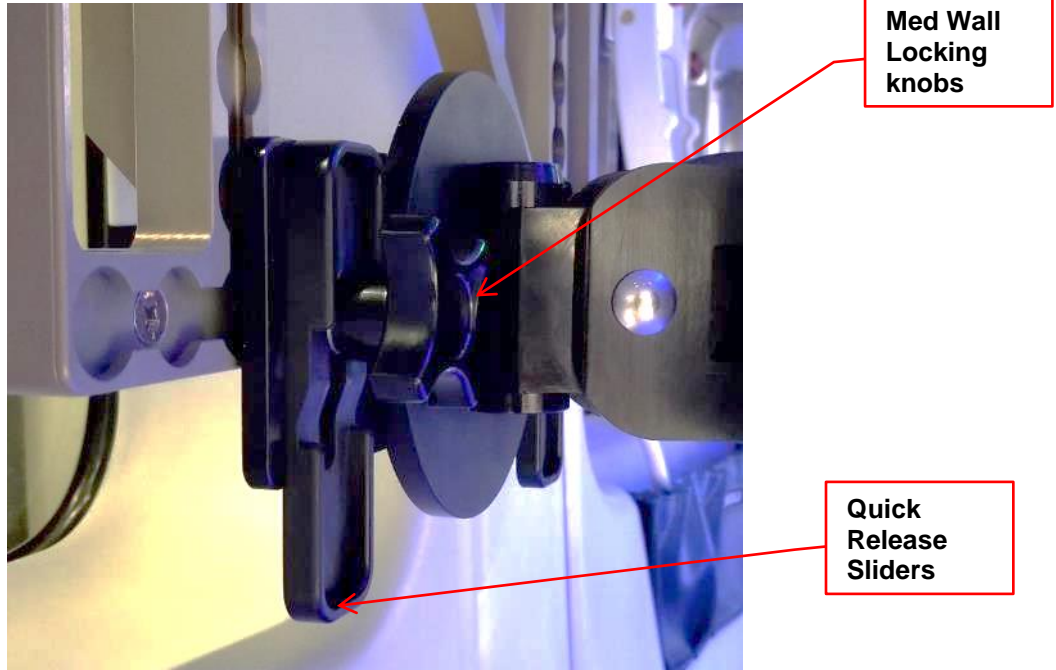

IPRS Main Component

- 1) To remove the infusion pump from IPRS lift plunger, turn plunger cap 90° and remove pump.

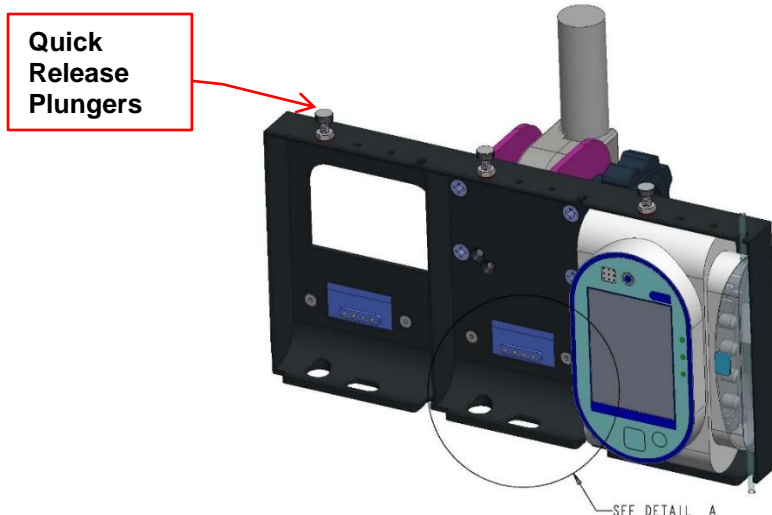



JJASPP ENGINEERING SERVICES, LLC  
JJASPP Medical Device Restraint System  
JJASPP-IP10-5/8 Data Sheet



JJASPP-IP30 Mount  
System with Swivel Rod



Medical Wall Attachment Component

- 1) To un-install Medical Wall Attachment Component from Med Wall, push both quick release slider up; then squeeze locking knobs and swivel plate together while pulling the swivel plate toward you. Then slide the Medical Wall Attachment Component left or right until it disengages from the Med Wall.
- 2) To install Main Component onto the Medical Wall Attachment Component, squeeze locking knobs and swivel plate together while pushing the swivel plate track studs into the Med Wall track. Then slide the Medical Wall Attachment Component left or right until it stops in the Med Wall. Press both quick release sliders down to lock the Attachment Component to the Med Wall.

**JJASPP-IP10-08**  
**Sapphire Fairfield Mounting System**

