



JJASPP ENGINEERING SERVICES, LLC  
JJASPP Medical Device Restraint System  
JJASPP-IP10-9 Data Sheet



## B Braun Triple Infusion Pump Mounting System



### Specifications:

Weight – 1.5 lbs.

Dimension – W:12.25", H: 3.81", D: 4.42"

### Certification:

The JJASPP MDRS (JJASPP-IP10-9) has FAA approval certification in the following aircraft platforms:

- 1) Airbus BK-117 series
- 2) Airbus EC155 series
- 3) Airbus EC135 series
- 4) Airbus EC130 series
- 5) Airbus AS350 series
- 6) Agusta A119, A109 series
- 7) Agusta AW119 MKII
- 8) Bell 206, 407, & 429 series
- 9) Pilatus PC-12 series
- 10) Beechcraft B200 series



**JJASPP ENGINEERING SERVICES, LLC**  
**JJASPP Medical Device Restraint System**  
**JJASPP-IP10-9 Data Sheet**

IP10-9 installed on FAA Approved EMS Medical Wall



IP10-9 without CME Infusion Pump Installed






# JJASPP ENGINEERING SERVICES, LLC

## JJASPP Medical Device Restraint System

### JJASPP-IP10-9 Data Sheet

#### Swivel Rod System (SRS) Main Housing Connection

SRS gives a 30° Vertical Range rotation capability and a 45° Horizontal Range rotation capability and a 180° Vertical tilting capability.

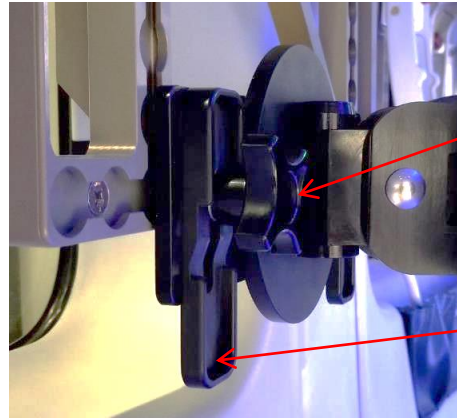

Med Wall  
Locking  
knobs

Quick  
Release  
Sliders

- 1) To remove the infusion pumps from IP10-9 pull plungers on both sides of the main housing, turn ring plungers 90° and remove pump.



JJASPP-IP30 Mount  
with Swivel Rod System  
(SRS)



#### Medical Wall Attachment Component

- 1) To un-install SRS from Med Wall, push both quick release slider up; then squeeze locking knobs and swivel plate together while pulling the swivel plate toward you. Then slide the SRS left or right until it disengages from the Med Wall.
- 2) To install the SRS, squeeze locking knobs and swivel plate together while pushing the swivel plate track studs into the Med Wall track. Then slide the SRS left or right until it stops in the Med Wall. Press both quick release sliders down to lock the SRS to the Med Wall.



**JJASPP ENGINEERING SERVICES, LLC**  
**JJASPP Medical Device Restraint System**  
**JJASPP-IP10-9 Data Sheet**

**JJASPP-IP10-9**  
**B Braun Mounting System**



**INFUSION PUMP Installation**  
**Into the SRS or to a Air Medical Pole**  
**Instructions**

1. Lift the quick release ring plungers on both sides of the main cage to the open position.
2. Install the pump fully into the IP10-9
3. Turn the quick release ring plungers 90°
4. The plunger pin will pop down into the pump slot on the backside of the pump.

